

Bio Systems User Guide - 30.2 litre containers

Bio Systems containers are reusable and are subject to different closing mechanisms than single use containers; please follow the instructions below.



OPEN

1. Firstly, slide the red locking mechanism to the rear of the container. Then engage the trolley's opening device by hooking the yellow lip around the red lock of the bio systems container.

RISK WASTE DISPOSAL

2. Firmly press on the foot pedal to open the lid.
3. Drop risk waste vertically into the container.
4. All waste that was previously disposed of into the yellow lidded rigid container and the blue or orange top sharps box can be disposed of into the bio systems 30 litre container.

To activate closure press down firmly on the red lock to engage locking mechanism. To reopen the lid, slide the red locking mechanism to the rear of the container.

FINAL CLOSURE

5. Fill to the mark fill line, never overfill. Do not sign or date the container. Add a cable tie to the handle of the container for traceability. Press down firmly on the red tab to the locked position. Push the yellow frame forward and lift out the container, replace it with an empty one.



These containers **MUST NOT BE PLACED IN WHEELIE BINS**